## Clinisept+

**ENG** **Clinisept+ Prep & Procedure User Guide**

Please read this leaflet carefully to ensure that you use Clinisept+ Prep & Procedure correctly to enable it to deliver the best possible results. Clinisept+ Prep & Procedure is a gentle, but highly effective skin cleanser that cleans, calms and cares for the skin. It contains the latest hypochlorite technology to ensure rapid antimicrobial protection, but without stinging or irritation. And, because of its unique oxidising method of action, it is not susceptible to antimicrobial resistance. Clinisept+ is also an effective deodoriser.

**Clinisept+ Prep & Procedure** is unlike any other cleanser and represents a significant development in skin care. It provides optimal conditions for skin recovery in a 15 second wet contact time:

- Cleansing antimicrobial
- Non-irritant
- Dermatologically tested
- Skin neutral pH
- Non-cytotoxic
- No alcohol

**Clinisept+ Prep & Procedure** is suitable for use on normal and sensitive skin. For professional use only.

**Using Clinisept+ Prep & Procedure:** **Clinisept+ Prep & Procedure** should be applied to clean skin. Before application, use a suitable make-up remover to remove any make-up or contaminants. Apply Clinisept+ to the whole area to cleanse the skin prior to any cosmetic or aesthetic treatment. An aseptic procedure should always be followed.

**Clinisept+ Prep & Procedure** can be applied directly from the bottle onto a sterile gauze or cleansing pad or can be poured into a clean gallipot. If using a gallipot, ensure a clean gallipot is used for each patient and discard any unused Clinisept+. Apply **Clinisept+ Prep & Procedure** by wetting a sterile gauze or cleansing pad with Clinisept+ and then gently wiping over the area using a circular motion. (1) 10 ml is sufficient to fully saturate 1 standard 10cm cotton gauze pad. Recommended volume per treatment 30 – 50 ml. The volume of Clinisept+ applied should be adjusted to suit the size of the area to be cleaned. In all instances, ensure the whole area is thoroughly wetted. Sterile gauzes and pads should be replaced and discarded as necessary. If applying Clinisept+ around the eyes, ask the client to close their eyes before wiping onto the eyelids. Clinisept+ is rapid in action and requires a contact time of only 15 seconds. Once applied, **Clinisept+ Prep & Procedure** leaves no residue on the skin and can be left to dry naturally. Do not dry or touch the skin after application unless with clean hands or sterile gloves.

**Cleansing before, during and after a procedure:** A skin neutral pH and non-cytotoxic chemistry allows Clinisept+ to be used before, during and after a treatment, thereby enabling high levels of hygiene to be maintained throughout a procedure. Use fresh sterile gauzes or cleansing pads as required and discard used pads. **Clinisept+ Prep & Procedure** provides a cooling and calming sensation and is very effective in reducing redness. For maximum care of recovering or irritated skin, (ie. immediately following a treatment or procedure), thoroughly soak a sterile gauze or cleansing pad with Clinisept+ and leave it placed on the affected area for five to ten minutes(2) 10 ml is sufficient to fully saturate 1 standard cotton gauze face masque. If the masque becomes dry, use a saturated gauze pad to replace more Clinisept+. Repeat as necessary.

**Cautions:**

Always follow the instructions carefully. For external use only. Avoid contact with eyes. If contact with eyes occurs, rinse with running water. Do not ingest. If ingested, seek medical advice as a precautionary measure. Clinisept+ has been dermatologically tested and proven to be non-irritant. Cease usage if irritation occurs. Avoid direct contact with clothing as may discolour fabric. This product requires no specific disposal procedure. Use within 3 months of opening. Avoid extremes of temperature and keep out of direct sunlight. Clinisept+ may be decanted into a gallipot for immediate use but do not decant into another container. All components fully recyclable – please recycle responsibly.

*Made in the UK*  
Clinical Health Technologies Ltd, Hinckley, LE10 3BP

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**Ingredients:** Wasser (Aqua), Phosphoric Acid, Sodium Hypochlorite. Contains no alcohol or fragrance and is colourant free.

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### FR Clinisept+ Prep & Procedure - Instructions

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