

## Cautions:

For single customer use only. For external use only. Avoid direct contact with clothing as may discolour fabric. Avoid contact with eyes. If contact with eyes occurs, rinse with running water. Do not ingest. If ingested, seek medical advice as a precautionary measure. Clinisept+ has been dermatologically tested and proven to be non-irritant. Cease usage if irritation occurs. This product requires no specific disposal procedure. Use within 3 months of opening. Do not decant from this bottle. Avoid extremes of temperature and keep out of direct sunlight. All components fully recyclable - please recycle responsibly.

## Ingredients:

Water(Aqua), Sodium Hypochlorite, Phosphoric Acid

**Contains no chlorhexidine or alcohol and is fragrance and colourant free.**



**Clinisept+ Procedure Aftercare** is one of a number of products in the Clinisept+ range.



*Made in the UK*

Clinical Health Technologies Ltd,  
Hinckley, LE10 3BP

+44(0)1733 232416

[www.cliniseptplus.com](http://www.cliniseptplus.com)

# Clinisept+

## Procedure Aftercare

Fast Acting Skin Cleanser  
& Antimicrobial Solution

Dermatologically Tested

## User Guide

## Clinisept+ Procedure Aftercare

**Clinisept+ Procedure Aftercare** has been recommended by your podiatrist to care for your feet. Please read this leaflet carefully to ensure that you use it correctly to enable it to deliver the best possible results.

### Clinisept+ Procedure Aftercare cleanses, calms and cares

for feet. It contains the latest hypochlorous technology to provide professional levels of antimicrobial protection without stinging or irritation. It provides optimal conditions for skin recovery:

- Cleansing antimicrobial
- Skin neutral pH
- Non-irritant
- Non-cytotoxic
- Dermatologically tested
- No chlorhexidine or alcohol

**Clinisept+ Procedure Aftercare** is suitable for use on normal and sensitive skin.

## Using Clinisept+ Procedure Aftercare:

Apply **Clinisept+ Procedure Aftercare** by spraying directly onto the intended area, or by wetting a gauze or cleansing pad with Clinisept+ and then gently wiping over the area. Ensure the area is **thoroughly** wetted at each application.



Once applied, **Clinisept+ Procedure Aftercare** leaves no residue on the skin and can be left to dry naturally. Do not dry the skin after application unless with a gauze, dressing pad, or clean towel if using at home.

To keep the skin cleansed and calm, Clinisept+ Procedure Aftercare should be applied 2-3 times a day as required, or as per instructed by your podiatrist.

**Clinisept+ Procedure Aftercare** provides a cooling and calming sensation and is effective in reducing the appearance of redness. For maximum care of recovering or irritated skin, spray regularly with Clinisept+ or thoroughly soak a gauze or pad with Clinisept+ and leave it placed on the affected area for five to ten minutes. Repeat as necessary.

